
CPR Bag (dark blue) & CPR-2 Bag (light blue) Disposable Manual Resuscitator Directions for Use (Adult, Small Adult, Child, Infant)

Notice: Should only be used by personnel trained in cardiopulmonary resuscitation.

Adult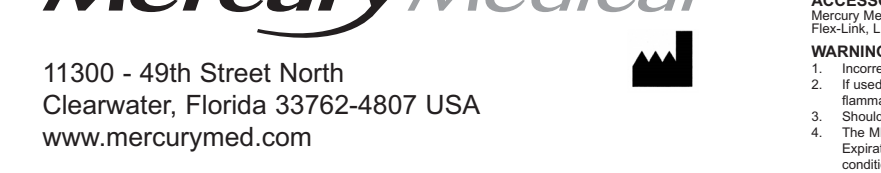


Small Adult


Child


Infant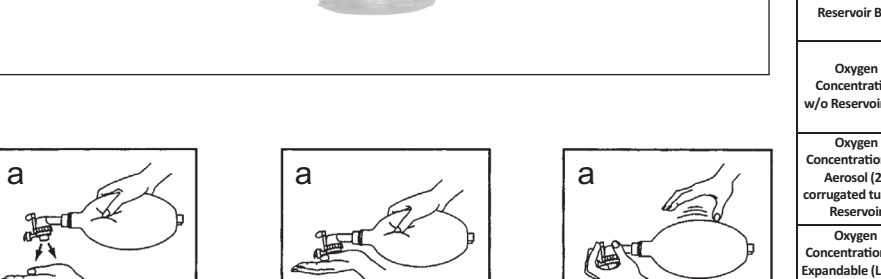


Child

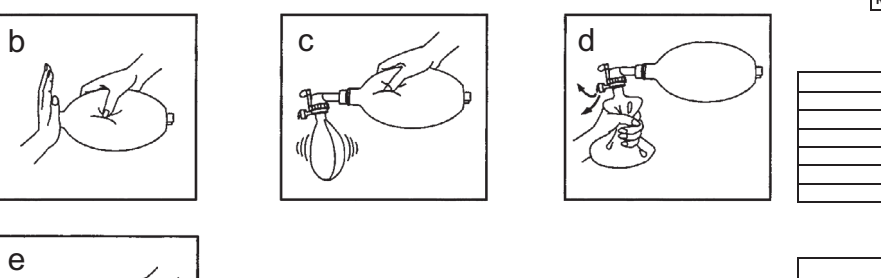Infant

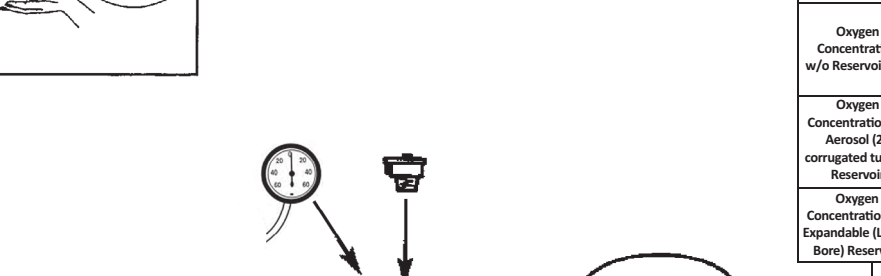Child

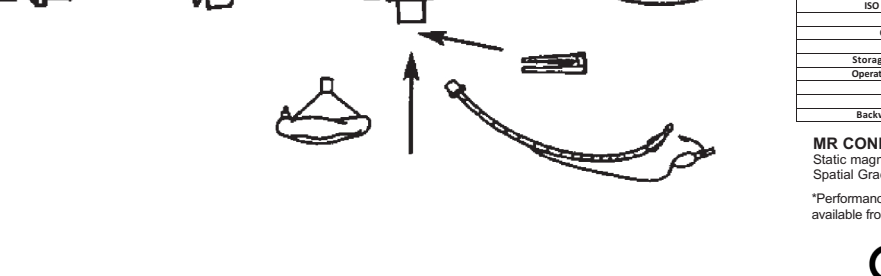Infant

CPR Bag (dark blue) & CPR-2 Bag (light blue) Disposable Manual Resuscitator Directions for Use (Adult, Small Adult, Child, Infant)

Notice: Should only be used by personnel trained in cardiopulmonary resuscitation.

INDICATIONS FOR USE: Provides emergency respiratory support by means of a face mask or a tube inserted into a patient's airway.

ACCESORIES: Mercury Medical® Manometer, PEEP Valve, OMM-LINK®, Face Masks, Filter, Disconnect Widge, PEEP Valve with Filter, Flex-Link, LifeSaver® Manometer, CO2 Detector

- 1. Insured operation of this device can be hazardous. 2. If used with supplemental oxygen, do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable vapors.

PRECAUTIONS: 1. Resuscitator should be used only by personnel trained in cardiopulmonary resuscitation.

DIRECTIONS FOR USE: 1. Remove the resuscitator from the outer protective poly bag. Expand the CPR-PR2 Bag to operating position.

ADULT: 1. Compress the resuscitator bag with one hand, then occlude patient valve outlet with your other hand.

SMALL ADULT: 1. Compress the resuscitator bag with one hand, then occlude patient valve outlet with your other hand.

CHILD: 1. Compress the resuscitator bag with one hand, then occlude patient valve outlet with your other hand.

INFANT: 1. Compress the resuscitator bag with one hand, then occlude patient valve outlet with your other hand.

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