

Airtraq

GUIDED VIDEO INTUBATION



Simple, effective,
affordable video
laryngoscopy

Exclusively Distributed by:

Mercury Medical[®]
Superior Solutions

(800) 835.6633

mercurymed.com

Two complementary models for one complete solution



AIRTRAQ AVANT

Reusable Optics + Disposable Blades

Minimum cost per intubation.
VL for frequent use.

AIRTRAQ SP

Fully Disposable


One piece. Ready to use.
VL for remote locations
and emergencies.




1. Slide midline around the tongue



2. Center vocal cords

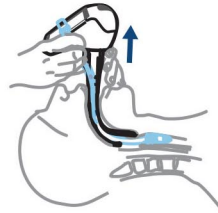

3. Push ETT

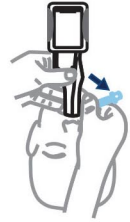

4. Remove midline

- * Keep Midline
- * Do not insert too deep



- * Lift gently
- * Twist Airtraq to center vocal cords
- * Corkscrew ETT if needed

Airtraq improves routine intubations and it is specially effective for:

- ➔ Cervical immobilization
- ➔ Obese
- ➔ Awake
- ➔ In combination with a fiberscope
- ➔ Frontal or lateral intubations
- ➔ Poor dentition

Airtraq sp

Adults
Pediatrics
Double Lumen
Nasal (no channel)



Adult regular
7.0-8.5 ETT
M.O.*: 16 mm
LDA011



Adult small
6.0-7.5 ETT
M.O.*: 15 mm
LDA021



Double lumen
28-41 Fr
M.O.*: 18 mm
LDA071



Nasal
M.O.*: 15 mm
LDA061

NOW
4X the Image
Resolution

Wi-Fi
Camera



iOS / Android
Mac / Windows



Pediatric
4.0-5.5 ETT
M.O.*: 12 mm
LDA031



Infant
2.5-3.5 ETT
M.O.*: 11 mm
LDA041

LDA390KIT: Standard Hard Case (Blue), Camera, USB, Silicone Cover, Docking Station & Wall Charger

LDA390: Small Hard Case (Blue), Camera, USB & Silicone Cover

LDA390ACC: Soft Neoprene Bag, Docking Station & Wall Charger

Adult regular Adult small
7.0-8.5 ETT 6.0-7.5 ETT
M.O.*: 17 mm M.O.*: 17 mm

LDA501

LDA511

LDA521

LDA590



Reusable Optics

Disposable blades

Docking Station

Airtraq Avant

* Mouth opening

How Airtraq simplifies Video Intubation

- Airtraq's channel simplifies ETT insertion reducing intubation time.
- No stylet needed. Avoids potential injuries. Eliminates stylet costs.
- 90° shape minimizes hyperextension and reduces force required.
- Reduces hemodynamic reactivity.
- Facilitates intubation from any position (face to face, etc.).
- High intubation success rate in difficult airways.
- Easier to learn than other video laryngoscopes.


**Multiple
viewing
options**

**90° channel
simplifies
intubation**

**Minimum capital
investment
allows availability
in every location**

Visualization Options

for any Airtraq model

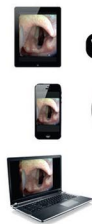

WiFi



Wi-Fi
Cam

Endo
Cams



Direct
View



Exclusively Distributed by:

MercuryMedical[®]
Superior Solutions

(800) 835.6633

mercurymed.com

Airtraq LLC
2414 Lawton Ln.
Rowlet, TX 75089, USA
Phone: 1-877-624-7929
E-mail: info.usa@airtraq.com

Airtraq
GUIDED VIDEO INTUBATION

www.airtraq.com

Prodol Meditec S.A
Muelle T. Olavarri, 5
48930 Las Arenas, Spain
Phone: +34 944 804 690
E-mail: info@airtraq.com