Mercury Medical[®] NIFometer[®]
Single-Patient-Use
Negative Inspiratory Force Meter

11300 - 49th Street North
Clearwater, Florida, 33762-4807 USA
www.mercurymed.com

DO NOT STERILIZE.

INDICATIONS FOR USE:
The Mercury Medical[®] NIFometer is used to indicate negative inspiratory force. Federal law (USA) restricts this device to sale by or on the order of a physician.

DIRECTIONS FOR USE:

1. Set NIFometer gauge to zero.
2. Connect the NIFometer to the endotracheal tube or elbow if using kit with mouthpiece.
3. To record the maximum negative inspiratory force, press down on the occlusion knob, then instruct patient to inhale. Air flow will be willfully occluded by clinician to record negative inspiratory force (see diagram).
4. Release knob immediately after measurement.
5. Memory pointer identifies and remains at the maximum inspiratory force to -40cm H₂O or to -60cm H₂O.
6. To reset memory pointer, rotate memory pointer knob counterclockwise.
7. When measurement is complete, remove NIFometer from endotracheal tube and re-connect breathing circuit per manufacturer's instructions or facility protocol.

WARNINGS:

- To create and measure negative inspiratory force, air flow must be willfully occluded by clinician. Once measurement is taken, release occlusion knob to allow air flow.
- Memory pointer must be set at zero before use.

CAUTIONS:

- Federal law (USA) restricts this device to sale by or on the order of a physician.
- During use, the patient's condition must be continuously monitored.
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.

SPECIFICATIONS:

Range: To -40cm H₂O or to -60cm H₂O
Variance: ± 2cm H₂O -20 to -30cm H₂O
Fitting: 15mm I.D./22mm O.D.
Weight: 30 grams nominal
Packaged: 20/Box
10 Kits/Box

Available as "unit only" for use with endotracheal tubes or in a kit with mouthpiece, elbow and nose clip.

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