**DESCRIPTION:**


The Mercury Medical® OMNI-LINK® or FLEX-LINK® was designed to extend the patient valve on the Mercury® CPR Bag Series and not add additional dead space to the system. The OMNI-LINK® or FLEX-LINK® absorbs movement, provides flexibility and accommodates a long reach in compromising situations.

**DIRECTIONS FOR USE:**

1. For Omni-Link: To use, remove the patient valve from the CPR Bag Series. See *Figure 1*.
2. Insert the male connector of the OMNI-LINK® or FLEX-LINK® into the bag and insert the patient valve in the female connector. See *Figure 2*.
3. Make sure fittings are secure before use.
4. Flex-Link can be expanded by pulling from both sides simultaneously.


**WARNING:**

- Do not insert the patient swivel connector (patient port) into the OMNI-LINK® or FLEX-LINK®.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.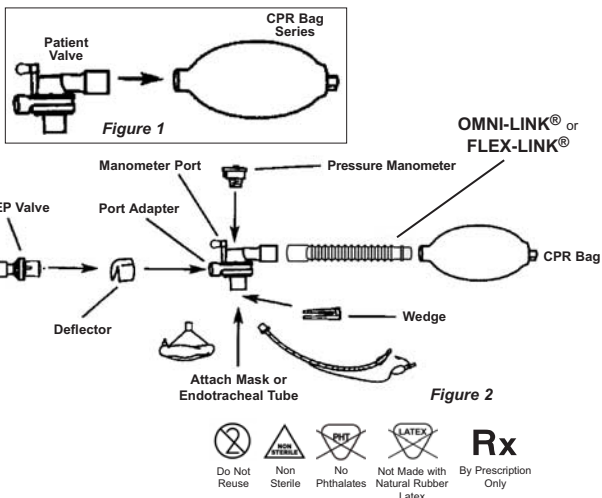

®Registered U.S. Trademark of Mercury Enterprises, Inc.

Assembled in USA with foreign components 3/2020 #83-200-0060 Rev. 5

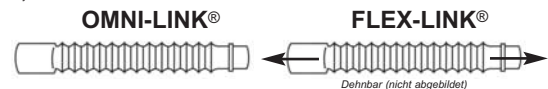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®Marca comercial registrada en EE. UU. de Mercury Enterprises, Inc.
®Registrerat USA- varumärke för Mercury Enterprises, Inc.